



# DIAGNOSTIC AUTOMATION, INC.

23961 Craftsman Road, Suite D/E/F, Calabasas, CA 91302

Tel: (818) 591-3030 Fax: (818) 591-8383

[onestep@rapidtest.com](mailto:onestep@rapidtest.com)

[technicalsupport@rapidtest.com](mailto:technicalsupport@rapidtest.com)

[www.rapidtest.com](http://www.rapidtest.com)



See external label



2°C-8°C



Σ=96 tests



Cat # 1804-09

## Total Human IgM Assay

Cat # 1804-09

Test	Total Human IgA Assay
Method	Enzyme Linked Immunosorbent
Principle	Peroxidase – Conjugated ELISA
Detection Range	0.031-2.0µg/mL
Sample	10µl serum
Specificity	N/A
Sensitivity	0.05 µg/mL
Total Time	~150min
Shelf Life	12-14 months

*\* Laboratory results can never be the only base of a medical report. The patient history and further tests have to be taken into account*

## Intended Use

To quantitate total human Immunoglobulin M (IgM)

## Principle of Procedure

Solid phase capture sandwich ELISA assay using a microwell format.

## Shelf Life

The expiration date for the package and each component is stated on the label(s). Store components at 2°-8°C, except for standard, which should be stored at -20°C.

## Patient and Standard Dilutions

Dilute each serum or plasma specimen to be tested initially 1:1000 in phosphate buffered saline (PBS), *e.g. 10ul of specimen into 10 ml of PBS*, then subdilute 1:10 with the IgM specimen diluent provided for a final dilution of 1:10,000. Prepare serial two fold dilutions of the human IgM standard: Neat, 1:2, 1:4, 1:8 etc. with the specimen diluent provided. Use the specimen diluent alone as the blank control well.

## Materials Supplied

1. Anti-Human IgM coated microwell strips 12x8 with plastic frame
2. HRP conjugated goat anti-human IgM -12mL
3. IgM standard (pre-diluted 1: 10,000)
4. TMB/peroxide substrate color developer -12mL
5. IgM specimen diluent (Specimen Diluent Green II) -1 x 60mL
6. Sulfuric acid termination reagent (0.5N) -12mL
7. 15 X Wash buffer concentrate - 60mL

## Limitations of the Procedure

No single assay should be used as the only basis for arriving at a diagnostic conclusion. For research use only.

## Dynamic Range

0.031µg/mL - 2.0µg/mL.

## Reproducibility

C.V. 6%-10% depending upon the region of the standard curve

## Assay Procedure


\*Allow each reagent to reach room temperature before use

1. Add 100uL of *diluted* specimen or standard to each microwell
2. Incubate at room temperature for 60 minutes
3. Decant and wash each microwell four times with wash buffer (dilute buffer 1:15 with reagent grade water)
4. Add 100uL of HRP conjugated goat anti-human IgM to each well
5. Incubate at room temperature for 60 minutes
6. Decant and wash as in step 3
7. Add 100uL of TMB/peroxide substrate and incubate at room temperature for 30 minutes

8. Terminate the reaction with 100uL of 0.5N sulfuric acid
9. Zero the microwell reader at 450nm using the specimen diluent zero control well
10. Determine the optical density (O.D.) of the remaining wells
11. Construct a standard curve using the O.D. values obtained for each of the standards
12. Interpolate the unknowns from the standard curve

**Note: Constructed 4 Parameter Curve displayed.**

### Typical Standard Curve

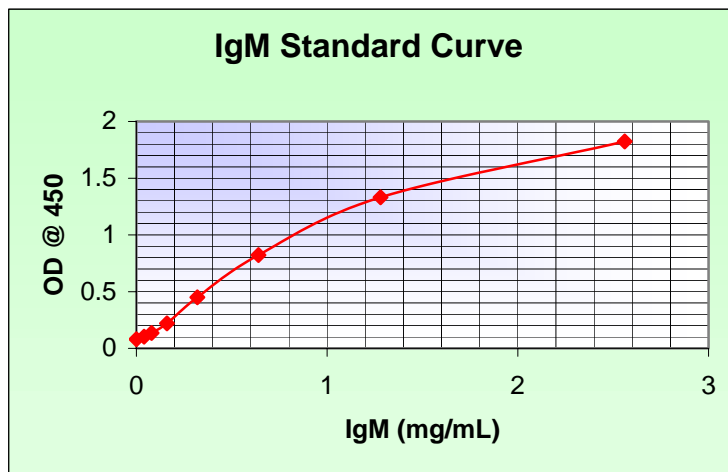

Date Adopted	Reference No.
2005-09-27	DA-Total Human IgM Assay-2009



**DIAGNOSTIC AUTOMATION, INC.**

23961 Craftsman Road, Suite D/E/F, Calabasas, CA 91302

Tel: (818) 591-3030 Fax: (818) 591-8383

**ISO 13485-2003**



Revision Date: 10/10/2009